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DE

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Instructions for Use

EN

This Instructions for Use is intended for use by lay person and health care professional.

Read the Instruction for Use before starting the treatment with Otinova.

Composition

Otinova solution contains aluminium acetate and aluminium acetotartrate (1.8 % aluminium), acetic acid and water (pH 3-4). The ear spray bottle contains 15 ml of the solution (150 doses).

Intended Purpose

Otinova is intended for treatment of ear canal inflammation, also known as Swimmer's Ear or external otitis.

Clinical Benefits

The clinical patient benefit to be expected of Otinova includes treatment of ear canal inflammation.

How does Otinova® work?

Otinova acts locally in the ear as a sprayed solution with antibacterial, antifungal, astringent and itch relieving effect.

How to use Otinova®?

1. Before Otinova is used for the first time, pump a few times until an even spray is obtained. Avoid spraying towards the face.
2. Hold the bottle upright.
3. Hold the tip of the nozzle against the outer ear canal.
4. Spray once or twice in the ear canal.
5. Tilt the head after spraying so that the fluid runs down into the ear canal.
6. Wipe away any excess fluid.
7. Repeat steps 2-6, morning and evening, for maximum 7 days.

Illustrations of Instruction for Use are found at www.otinova.com.

Warnings and Precautions

Otinova should not be used if the eardrum has been fitted with a tube or if the eardrum is perforated. Otinova should not be used in children younger than 5 years of age unless recommended by a healthcare professional (after control of the eardrum). Avoid contact with the eyes. If the fluid gets into the eyes, rinse thoroughly with water. Avoid inhaling the spray. Store out of reach of children. Contact your physician if there is no significant improvement after 3 days of treatment. Do not use Otinova for a continuous period longer than 7 days. Wait at least one week after ending a treatment period until a new treatment period is started. Contact your physician if there is marked discomfort, tenderness in and around the ear, if hearing is impaired, or if there is irritation/stinging sensation in the ear that persists or becomes worse.

Side effects

Otinova may cause a temporary mild irritation/stinging sensation in the ear canal. Rare cases of local hypersensitivity have been reported. Otinova may cause salt precipitation in the ear canal. Rare cases of perforated ear drum have occurred in association with extreme salt precipitation at high doses exceeding maximum recommended treatment period.

Notice to User

Otinova can be used by adults and children aged 5 years and up. Otinova is intended for single person use.

Any serious incident that has occurred in relation to the device should be reported to Circius Pharma AB (info@CirciusPharma.com) or the local competent authority.

Storage and Disposal

Store at room temperature.

Use within six months after opening.

Do not use after the expiry date.

Recycling according to local environmental procedures.

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